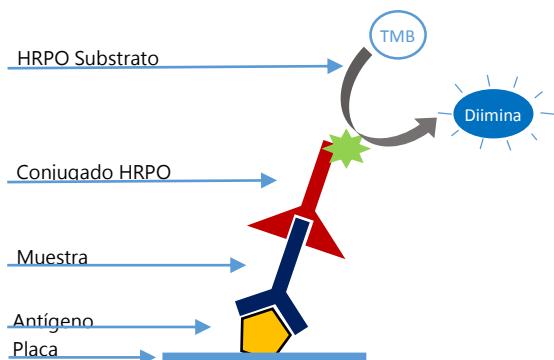
INgezim Brucella Porcina

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
Ed.020217

INgezim Brucella Porcina

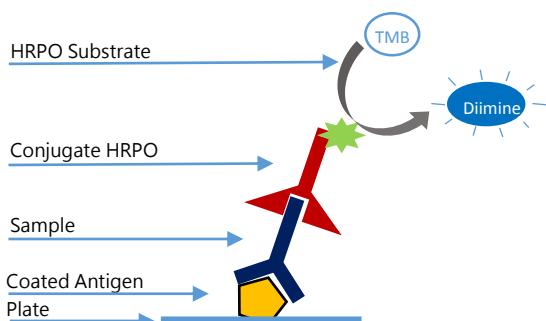INgezim Brucella porcina is an immunoenzymatic assay based in indirect ELISA technique, which uses a monoclonal antibody (MAb) specific for porcine IgG immunoglobulins and Lipopolisacaride (SLPS) from *smooth brucella*.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with inactivated smooth *Brucella* antigen (LPS). Samples are added to the wells and incubated.
2. If the sample contains antibodies to *Brucella*, they will bind to the antigen.
3. To detect this reaction a MAb-PO specific of porcine IgG is added. This binding is detected by colorimetric reaction after the addition of the substrate.

APPLICATION

Detection of specific antibodies to LPS of smooth *Brucella* in porcine serum samples.

INTERPRETATION OF THE RESULTS

For results interpretation, one cut off is used: Samples will be considered **Positive** if their OD values are equal or higher than the cut off. Samples will be considered **Negative** if their OD values are lower than the negative cut off.

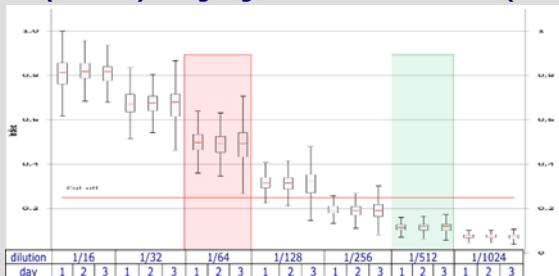
VALIDATION

1. Use of OIE Reference Serum.

In order to validate the assay respect of OIE regulation, the Reference Serum: "Suis International Standard Serum" was used. The assay is able to detect it as positive at 1/16 dilution fulfilling the OIE requirements.

2. Standardization of the new European Reference Serum for porcine Brucellosis (EUPBSS) using Ingezim® Brucella Porcina (7th workshop of the EU Brucellosis NRLs. 2014)

This assay has been used in 2014 in the Ring Test organized by ANSES for validation of the EUPBsSS. In this study participated 23 laboratories; 9 of them tested also the serum ISaBsS. Serum EUPBsSS was tested at dilutions in fold of 2, starting at 1/16 to 1/1024. The study allow to reach the following conclusions: Setting of the cut off positive/ negative for EUBsSS in 1/64 / 1/512 respectively, There Is not significant variability inter day (3 days), neither inter sample using INgezim® Brucella porcina.

3. Sensitivity and specificity.


Study 1. (AFFSA) A panel of 998 positive and negative sera previously classified by Rose of Bengale (RBT) and the competition ELISA INgezim Brucella Compac 2.0 was analysed. Results obtained indicated 92.38% correspondence with RBT and 97.8% with INgezim Brucella Compac. 2.0.

Study 2. (Unidad de Sanidad Animal. CITA. Zaragoza. Spain) The following gold standard sera were used: 88 domestic pigs from Brucella free herds, 62 domestic pigs naturally infected with *B. suis* biovar 2 and 20 wild boars naturally infected with *B. suis* biovar. (these 82 positive sera were confirmed by individual bacteriology). The pig sera were analysed by RBT and Complement Fixation (CF). Moreover, with the results obtained with these 3 groups, a ROC analysis was performed. **The assay showed 100% sensitivity and specificity.**

| | RBT | CFT ^a | Ingezim B. PORCINA |
|-----------|-----------|------------------|--------------------|
| Culture + | 62 | 59 | 62 |
| POSITIVE | 62 (100%) | 28 (47.4%) | 62 (100%) |
| NEGATIVE | 0 | 4 | 0 |

| | RBT | CFT ^a | Ingezim B. PORCINA |
|----------|-----------|------------------|--------------------|
| Free | 88 | 62 | 88 |
| POSITIVE | 0 | 0 | 0 |
| NEGATIVE | 88 (100%) | 6 (9.7%) | 88 (100%) |

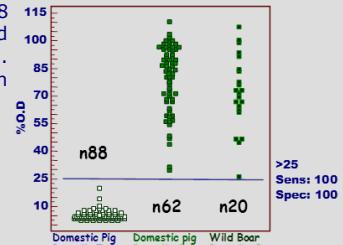^a A total of 27 sera from culture positive animals and 56 sera from *Brucella* free animals were showing anticomplementary activity.

Study 3. (Unidad de Sanidad Animal. CITA. Zaragoza. Spain). 555 sera from herds showing different status concerning *B. suis* were analysed. Groups 1 and 2 suffering reproductive problems, groups 3 and 4 not suffering reproductive problems.

| Group 1 (n 145. Isolation positive) | | | |
|-------------------------------------|---------------------------|-------------|----------------|
| | RBT | CFT | INgezim B. P. |
| POSITIVE | 132 (91%) ¹ | 90 (62%) | 120 (82.7%) |
| NEGATIVE | 13 | 48 | 25 |
| PA ² | 7 | | |

| Group 2 (n 134. Farms showing many RBT reactors) | | | |
|--|---------------|-------------|---------------|
| | RBT | CFT | INgezim B. P. |
| 43 (30.1%) | 19 (14.1%) | 51 (38%) | |
| 91 | 89 | 83 | |
| | 26 | | |

| Group 3 (n 60. Farms showing very few RBT reactors) | | | |
|---|-----------|------------|---------------|
| | RBT | CFT | INgezim B. P. |
| 3 (5%) | 0 (0%) | 6 (10%) | |
| 57 | 60 | 54 | |
| | 0 | | |

| Group 4 (n 216. Farms without RBT reactors) | | | |
|---|-----------|-------------|---------------|
| | RBT | CFT | INgezim B. P. |
| 0 (0%) | 0 (0%) | 7 (3.2%) | |
| 216 | 205 | 209 | |
| | 11 | | |

¹ Aparent prevalence percentage in each test; ² number of sera showing anticomplementary reactions.

ATTENTION: Cross-reaction in 2 of 66 sera of *Yersinia enterocolitica* infected animals has been detected.

COMPOSITION OF THE KIT

- Microtitration plates of 96 wells
- Vial with Positive Control.
- Vial with Negative Control.
- Vial with Conjugate.
- Bottles with Washing Solution.
- Bottles with diluent.
- Bottle with substrate (TMB) ready to use.
- Bottle with Stop Solution

SHELF LIFE:
15 months
Store at 2°C-8°C

PRODUCT MANUFACTURED BY INGENASA
REGISTRATION NUMBER: 3462 RD